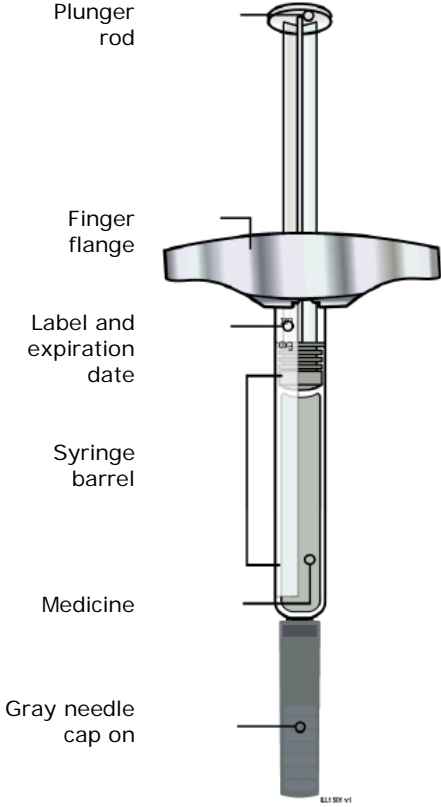
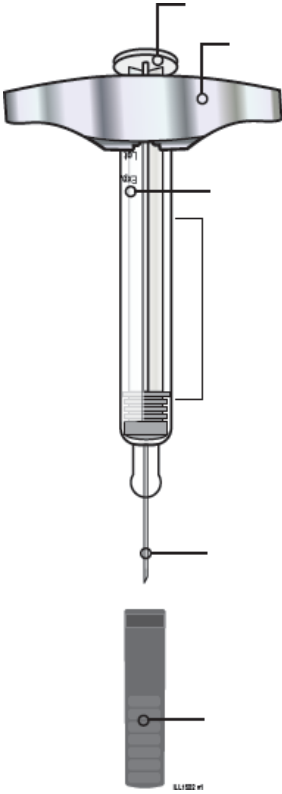


**Instructions for Use:**  
**EVENTITY® (romosozumab)**  
**Solution for Subcutaneous Use**  
**Single-Use Pre-filled Syringe**

**Guide to parts**

Before use	After use
 <p>Plunger rod</p> <p>Finger flange</p> <p>Label and expiration date</p> <p>Syringe barrel</p> <p>Medicine</p> <p>Gray needle cap on</p>	 <p>Used plunger rod</p> <p>Finger flange</p> <p>Label and expiration date</p> <p>Used syringe barrel</p> <p>Used needle</p> <p>Gray needle cap off</p>

**Important:** Needle is inside

---

## Important

Before you use an Evenity pre-filled syringe, read this important information:

**Use both syringes in this carton for one full dose.**

### Storing the Evenity pre-filled syringe

- Keep the syringe out of the reach of children.
- Keep the syringe in the original carton to protect from light or physical damage.
- The syringe should be kept in the refrigerator at 2°C to 8°C.
- If needed, you may store the syringe at room temperature (up to 25°C) for up to 30 days. Throw away Evenity that has been stored at room temperature after 30 days.
- **Do not** store the syringe in extreme heat or cold. For example, avoid storing in your vehicle's glove box or trunk.
- **Do not** freeze.

### Using the Evenity pre-filled syringe

- Administration should be performed by an individual who has been trained to administer Evenity.
- **Do not** use a syringe after the expiration date on the label.
- **Do not** shake the syringe.
- **Do not** remove the gray needle cap from the syringe until you are ready to inject.
- **Do not** freeze or use the syringe if it has been frozen.
- **Do not** use a syringe if it has been dropped on a hard surface. Part of the syringe may be broken even if you cannot see the break. Use a new syringe.
- The syringe is not made with natural rubber latex.


## Step 1: Prepare



**Read this before you inject.**

- To deliver a full dose, inject **two** 105 mg syringes, one after the other.


**A** Remove two syringes from the carton.

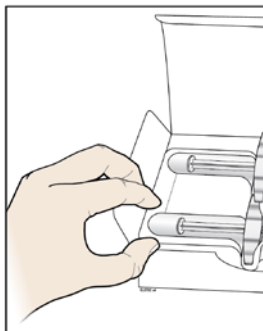

Place finger or thumb on edge of tray to secure it while you remove syringe.

Grab the syringe barrel to remove the syringe from the tray.

**Grab Here**

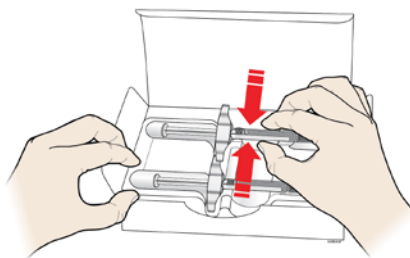

For safety reasons:


- Do not** grasp the plunger rod.
- Do not** grasp the gray needle cap.
- Do not** remove the gray needle cap until you are ready to inject.
- Do not** remove the finger flange. This is part of the syringe.

Leave the syringes at room temperature for at least 30 minutes before injecting.

- Do not** put the syringes back in the refrigerator once they have reached room temperature.
- Do not** try to warm the syringe by using a heat source such as hot water or microwave.
- Do not** leave the syringes in direct sunlight.
- Do not** shake the syringes.

**Important:** Always hold the pre-filled syringe by the syringe barrel.

## B Inspect the syringe.

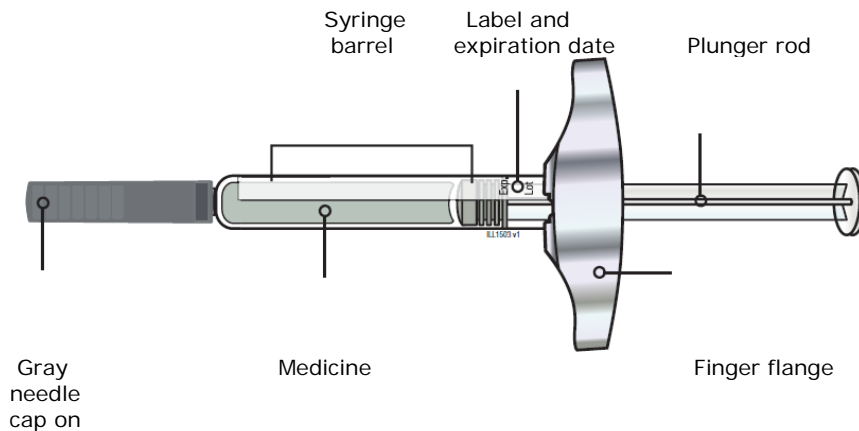

**Always hold the syringe by the syringe barrel.**

**Make sure the medicine in the syringe is clear and colourless to light yellow.**

- **Do not** use the syringe if the medicine is cloudy or discoloured or contains particles.
- **Do not** use the syringe if any part appears cracked or broken.
- **Do not** use the syringe if the gray needle cap is missing or not securely attached.
- **Do not** use the syringe if the expiration date printed on the label has passed.


In all cases, use a new syringe.

## C Gather all materials needed for the injection.


Wash your hands thoroughly with soap and water.

On a clean, well-lit work surface, place the:

- Two syringes
- Two alcohol wipes
- Two cotton balls or two gauze pads
- Two adhesive bandages
- Sharps disposal container

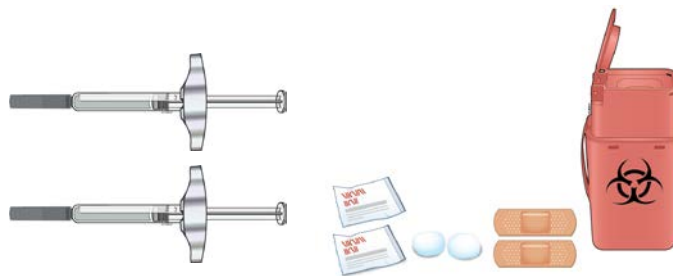


**D** Prepare and clean two injection sites, one for each of the two injections.

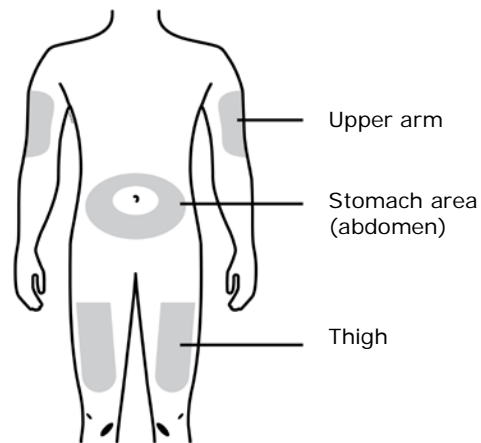

**You can use:**

- The thigh
- Stomach area (abdomen), except for a two-inch area right around the navel
- Outer area of upper arm

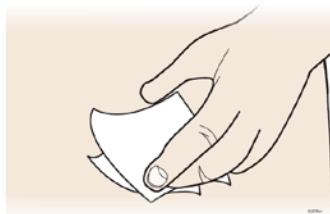


Clean the injection sites with alcohol wipes. Let the skin dry.

- **Do not** touch this area again before injecting.
- Choose a different site each time you give an injection. If you want to use the same injection site, make sure it is not the same spot on the injection site you used for a previous injection.
- **Do not** inject into areas where the skin is tender, bruised, red, or hard.
- Avoid injecting into raised, thick, red, or scaly skin patch or lesion, or areas with scars or stretch marks.

## Step 2: Get ready

- E** Choose the first syringe. Pull the gray needle cap straight off and away from the body when you are ready to inject.

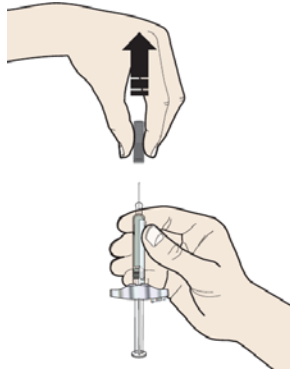

It is normal to see a drop of liquid at the end of the needle.

- **Do not** twist or bend the gray needle cap.
- **Do not** put the gray needle cap back onto the syringe.
- **Do not** remove the gray needle cap from the syringe until you are ready to inject.

**Important:** Throw the needle cap into the sharps disposal container provided.

- F** Pinch the injection site to create a firm surface.




Pinch skin firmly between your thumb and fingers, creating an area about two inches wide.

**Important:** Keep skin pinched while injecting.


### Step 3: Inject

- G** Hold the pinch. With the gray needle cap off insert the syringe into the skin at 45 to 90 degrees.

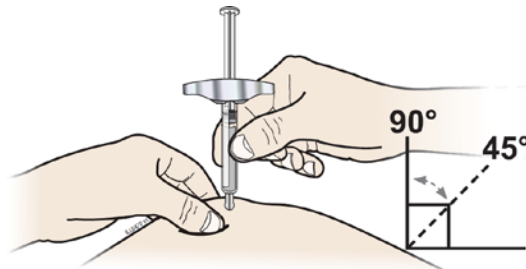


**Do not** place your finger on the plunger rod while inserting the needle.

- H** Using slow and constant pressure push the plunger rod all the way down until it stops moving.

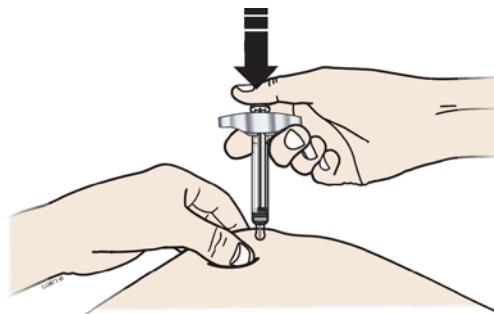

- I** When done, release your thumb and gently lift the syringe off of the skin.

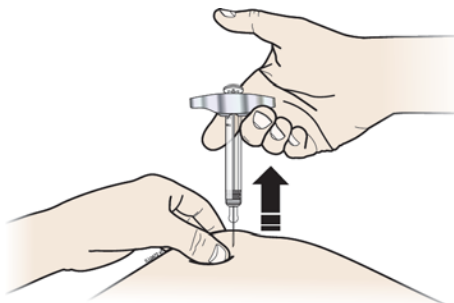

**Note:** After you remove the syringe from the skin, the syringe barrel should be empty.

**Important:** If it looks like the medicine is still in the syringe barrel, this means you have not delivered a full injection.

## Step 4: Finish

**J** Discard the used syringe and the gray needle cap.



Put the used syringe in a sharps disposal container right away after use. **Do not** throw away (dispose of) the syringe in your household trash.

If you do not have a sharps disposal container, you may use a household container that is:

- Made of a heavy-duty plastic
- Can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out
- Upright and stable during use
- Leak-resistant
- Properly labeled to warn of hazardous waste inside the container

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles and syringes.

- **Do not** reuse the syringe.
- **Do not** recycle the syringe or sharps disposal container or throw them into household trash.

**Important:** Always keep the sharps disposal container out of the reach of children.


**K** Examine the injection site.

If there is blood press a cotton ball or gauze pad on the injection site. **Do not** rub the injection site. Apply an adhesive bandage if needed.



---

**L** Repeat all steps with the second syringe to inject the full dose.

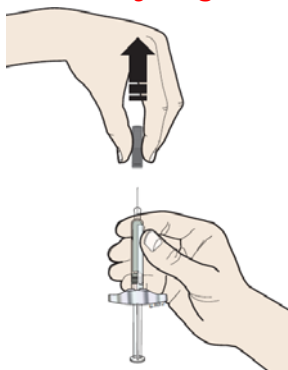

November 2019



EVENITY® is a registered trademark owned or licensed by Amgen Inc., its subsidiaries, or affiliates.

MYEVEIFU01